(GB)

**Pink paste opaque for covering model casts with incorporated adhesive**

**Instructions for use**

The surfaces of the framework should be finished and polished, and then cleaned of any grease and polish residue. Sandblast the surface with 50 micron aluminium oxide and steam and dry with oil-free air.

Tender pink paste opaque is applied with a stiff, short-haired brush. Incomplete opaque coverage negatively affects the shade. At least two coats are required to ensure complete coverage. The flow of the opaque is improved by mixing it on a mixing pad. The paste should be applied especially thinly. Each layer is polymerised separately.

Laborlux-L:	3 min
Lampada PlusT:	4 min
DC-XS/Uni-XS	90 sec
Triad II	5 min
Solidilite	1 min
MPA 2000	1xHD
Labolight LV-II	1 min
Spektra2000	3 min
Spektramat	2 min

NOTE: if the opaque layers are applied too thickly, it prevents optimum polymerisation and weakens the bond. The opaque should only be applied in very thin layers.

**Working time:** 1-3 min. depending on light conditions

**Storage:** <25°C. Close the syringe after use and protect from direct light.

Not use after expiry date.

**Side effects:** extremely rare

**Contraindications/interactions:** do not use in case of hypersensitivity of the patient to one of the components. Inform the dentist of the above information for a possible interaction with medicine and other materials present in the mouth.

MICERIUM Spa Via Marconi 83 - 16030 - Avegno (GE) Italy  
Tel. +39 0185 7887 870 Telefax: +39 0185 7887 970 <http://www.micerium.it>

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