Eigenschaften

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
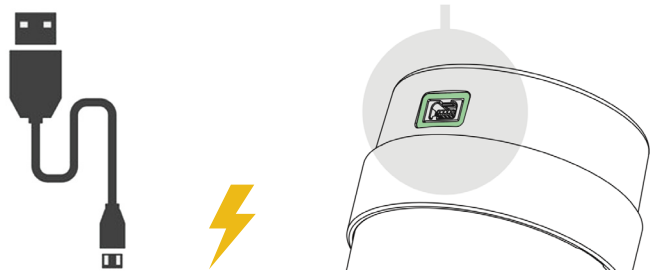
1. Fully charge the unit


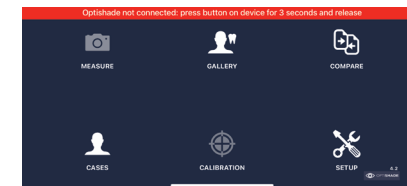
3. Connect Optishade to an iOS device and follow app instructions

2. Download free Optishade app

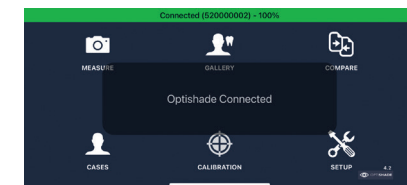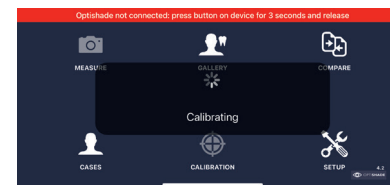4. Calibrate, remove the cap and start measuring

SPECIFICATIONS

Optishade is a colorimeter, an optoelectronic device, that allows to measure colors in the dental color space. Optishade works exclusively together with an iPhone or an iPad (not supplied).

VERY IMPORTANT: your Optishade is an advanced technology device that needs to be manipulated with the greatest care. Any issue resulting from a wrong or abusive use of your Optishade is not covered by the warranty (warranty: 1 year).

Unit dimensions	length 51 mm diameter 81,5 mm weight 60g
Functions with iOS	iPhone, iPad and iPod Touch Comes with the following two cables: - Lightning - USB

#70100-05 - 04/2021

Smile Line SA
Z.I. La Clef, 5
CH-2610 St. Imier
Switzerland
+41 32 944 22 44
mail@smileline.ch
www.smileline.ch

Contents

- Optishade unit complete with Capture Guide Cone and Calibration Cap
- 1 lightning cable for connection Optishade/iPhone-iPad - 1 USB Charging Cable
- 4 Capture Guide Cones
- 4 Calibration Caps
- 1 Instruction for use booklet and 1 quick start guide
- Optishade App (free) to be downloaded on Appstore
- 30 days free licence Matisse

NOT STERILE. REFER TO CLEANING AND DISINFECTING SECTION

WARNING! A calibration card is located inside the calibration cap. The calibration card is extremely fragile and sensible. You should in no way touch it with your fingers nor risk to stain it. The calibration Cap cannot be disinfected nor sterilized. In case you are unsure about its cleanliness or in case of a risk of cross contamination, please immediately discard it and take a new one.

WARNING! Each component has only one mounting position possible. It is recommended to train a little bit for a good understanding of mounting principles.

WARNING! Never try to open the Optishade base. This one is sealed and the fact of opening it cancels the warranty of the producer. **Beware** for not risking any infiltration of liquid inside the base of your Optishade.

VERY IMPORTANT! Any damage caused by an infiltration of liquid inside the base of your Optishade is not covered by the producer's warranty.

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